



# SeaMar<sup>®</sup> Nasal Hygiene

## BREATH EASY DAILY



### Category

---

Medical device

### Form

---

Spray

### Composition

---

**SeaMar<sup>®</sup> - Nasal Hygiene** solution contains:

Isotonic Sea Water solution, rich with the sea's beneficial ingredients (mineral salts and trace elements), purified water

### Indications

---

**SeaMar<sup>®</sup> - Nasal Hygiene** is indicated to cleanse the nasal cavities effectively. It is ideal for regular nasal hygiene

### Administration & Dosage

---

1-3 sprays per nostril, 2-3 times daily, or as per your physician's instructions

Blow nose before using **SeaMar<sup>®</sup> - Nasal Hygiene**. Bend neck gently forward over a washbasin and tilt your head to one side. Gently insert the nozzle into the opposite nostril, in parallel with the nasal septum, and press firmly. Turn head to the opposite side and repeat the application in the other nostril

### Storage

---


Keep at a temperature not exceeding 30 °c in a dry place.

Keep out of reach of children

### Package

---

Carton box contains Alu bottle with Blow off valve and SPI packing insert.

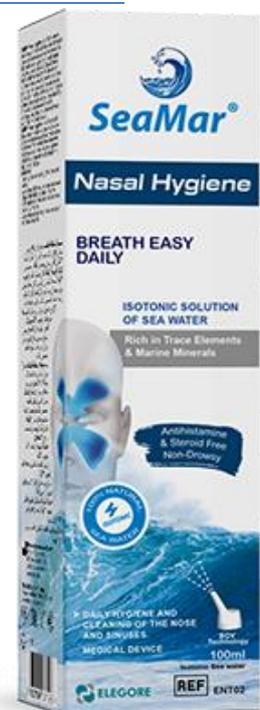



# SeaMar<sup>®</sup> Nasal Hygiene

## BREATH EASY DAILY



### Benefits & Properties

---

- **SeaMar<sup>®</sup> - Nasal Hygiene is a 100% natural, clinically proven nasal spray that relieves nasal congestion naturally, thins mucus, and cleanses the nasal cavities effectively, helping to protect from infections. It is ideal for regular nasal hygiene.**
- **SeaMar<sup>®</sup> - Nasal Hygiene decongests the nose and thins mucus making it easier to eliminate, restoring nasal breathing naturally.**
- **SeaMar<sup>®</sup> - Nasal Hygiene cleanses the nasal cavities effectively (continuous flow spray), removing mucus and mucus-trapped impurities (infectious agents, allergens, irritants, pollutants), Helps restore the natural function of the nose.**